

## ASEPT<sup>®</sup> Safety Biopsy Needle & Biopsy Needle Kits

Safety - the next  
breakthrough for soft  
tissue biopsy needles

[www.pfmmedical.com](http://www.pfmmedical.com)



The ASEPT® Safety Biopsy System is intended to be used for soft tissue and tumor biopsy.

### 3 Safety Features

1

#### **No exposed introducer Needle**

Introducer needle is placed. Then while removing needle from cannula, needle shaft is covered by tape while withdrawal needle and needle tip is locked inside cartridge.




Removing needle from introducer cannula.

2

#### **Finger guard protection for clinical holding introducer**

Finger guard is placed into introducer needle to prevent needle stick.

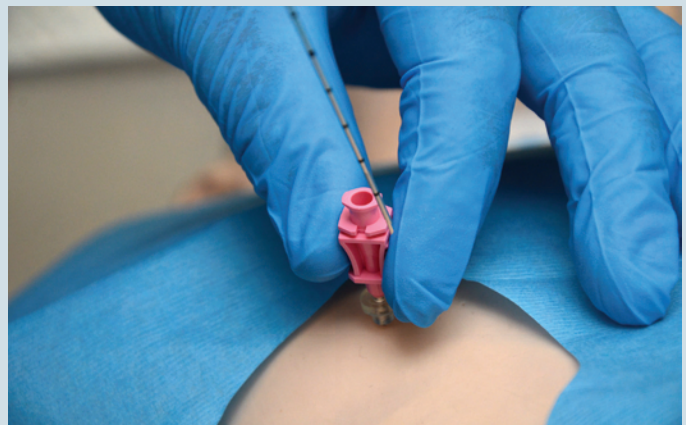


Introducer without finger protector.

3

#### **No touch contamination and no exposed needle tip to get rid of**

When finished with biopsy, needle is covered by tape in cartridge and needle tip is locked inside cartridge.

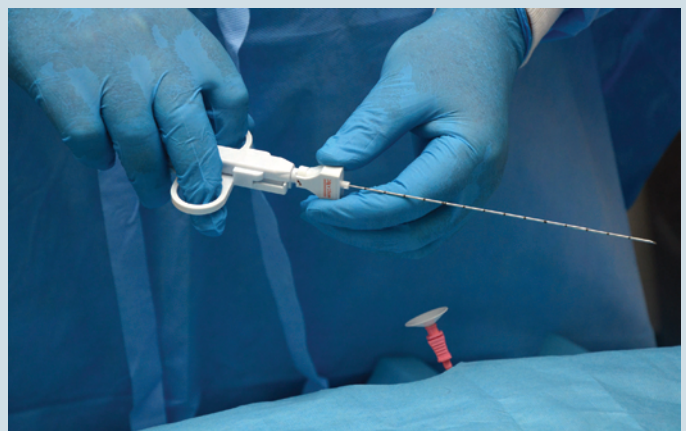


Turning cartridge to release safety.





**Pulling needle out with safety tape covering needle shaft.**

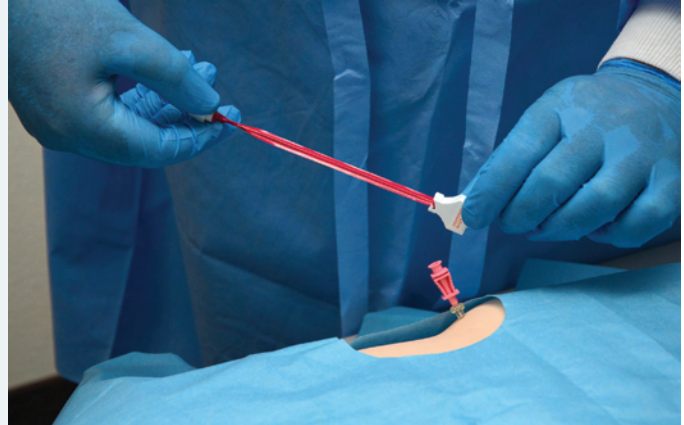

**Needle point locked inside cartridge.**




**Introducer with finger guard.**



**Biopsy Needle inside introducer cannula.**



**Pulling cartridge down to release tape covering.**

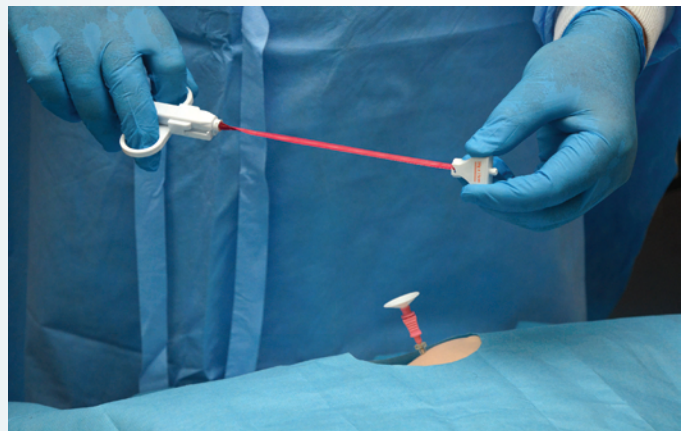

**Needle locked inside cartridge.**

## Disposable Semi-Automatic Biopsy Needle

Item No.	Needle Device	Case Qty
SBN 14-09	14g x 9cm, 2cm Notch, Bevel Tip	5/Case
SBN 14-15	14g x 15cm, 2cm Notch, Bevel Tip	5/Case
SBN 16-09	16g x 9cm, 2cm Notch, Bevel Tip	5/Case
SBN 16-15	16g x 15cm, 2cm Notch, Bevel Tip	5/Case
SBN 18-09	18g x 9cm, 2cm Notch, Bevel Tip	5/Case
SBN 18-15	18g x 15cm, 2cm Notch, Bevel Tip	5/Case
SBN 18-20	18g x 20cm, 2cm Notch, Bevel Tip	5/Case
SBN 20-09	20g x 9cm, 2cm Notch, Bevel Tip	5/Case
SBN 20-15	20g x 15cm, 2cm Notch, Bevel Tip	5/Case
SBN 20-20	20g x 20cm, 2cm Notch, Bevel Tip	5/Case

## Safety Coaxial Needle

Item No.	Needle Device	Case Qty
SCN 15-06	15g x 6cm Introducer w/Finger Guard	10/Case
SCN 15-12	15g x 12cm Introducer w/Finger Guard	10/Case
SCN 17-06	17g x 6cm Introducer w/Finger Guard	10/Case
SCN 17-12	17g x 12cm Introducer w/Finger Guard	10/Case
SCN 17-17	17g x 17cm Introducer w/Finger Guard	10/Case
SCN 19-06	19g x 6cm Introducer w/Finger Guard	10/Case
SCN 19-12	19g x 12cm Introducer w/Finger Guard	10/Case
SCN 19-17	19g x 17cm Introducer w/Finger Guard	10/Case

## Biopsy Needle with Safety Coaxial Needle Product Line

Item No.	Needle Device	Case Qty
SBCN 14-09K	14g x 9cm, 2cm Notch, Bevel Tip, 13g x 6cm Introducer	5/Case
SBCN 14-15K	14g x 15cm, 2cm Notch, Bevel Tip, 13g x 12cm Introducer	5/Case
SBCN 16-09K	16g x 9cm, 2cm Notch, Bevel Tip, 15g x 6cm Introducer	5/Case
SBCN 16-15K	16g x 15cm, 2cm Notch, Bevel Tip, 15g x 12cm Introducer	5/Case
SBCN 18-09K	18g x 9cm, 2cm Notch, Bevel Tip, 17g x 6cm Introducer	5/Case
SBCN 18-15K	18g x 15cm, 2cm Notch, Bevel Tip, 17g x 12cm Introducer	5/Case
SBCN 18-20K	18g x 20cm, 2cm Notch, Bevel Tip, 17g x 17cm Introducer	5/Case
SBCN 20-09K	20g x 9cm, 2cm Notch, Bevel Tip, 19g x 6cm Introducer	5/Case
SBCN 20-15K	20g x 15cm, 2cm Notch, Bevel Tip, 19g x 12cm Introducer	5/Case
SBCN 20-20K	20g x 20cm, 2cm Notch, Bevel Tip, 19g x 17cm Introducer	5/Case